




Quick Start Guide


step

1


Place your new ClearCaptions® Phone phone near a power outlet and a wired or wireless internet access point. If you'll be using your own phone service, your phone will also need to be located near a phone jack.

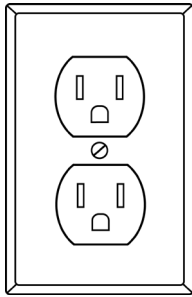

Power outlet

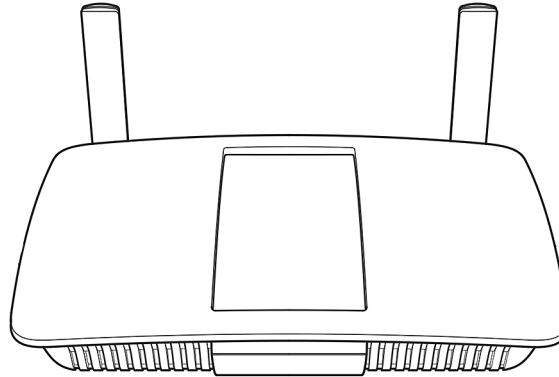

Wired or wireless router

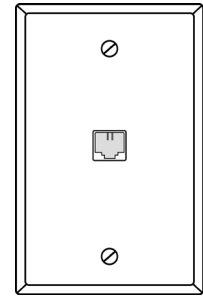


Phone jack

step

2

Remove the ClearCaptions Phone body, handset, cords and phone stand from the box.



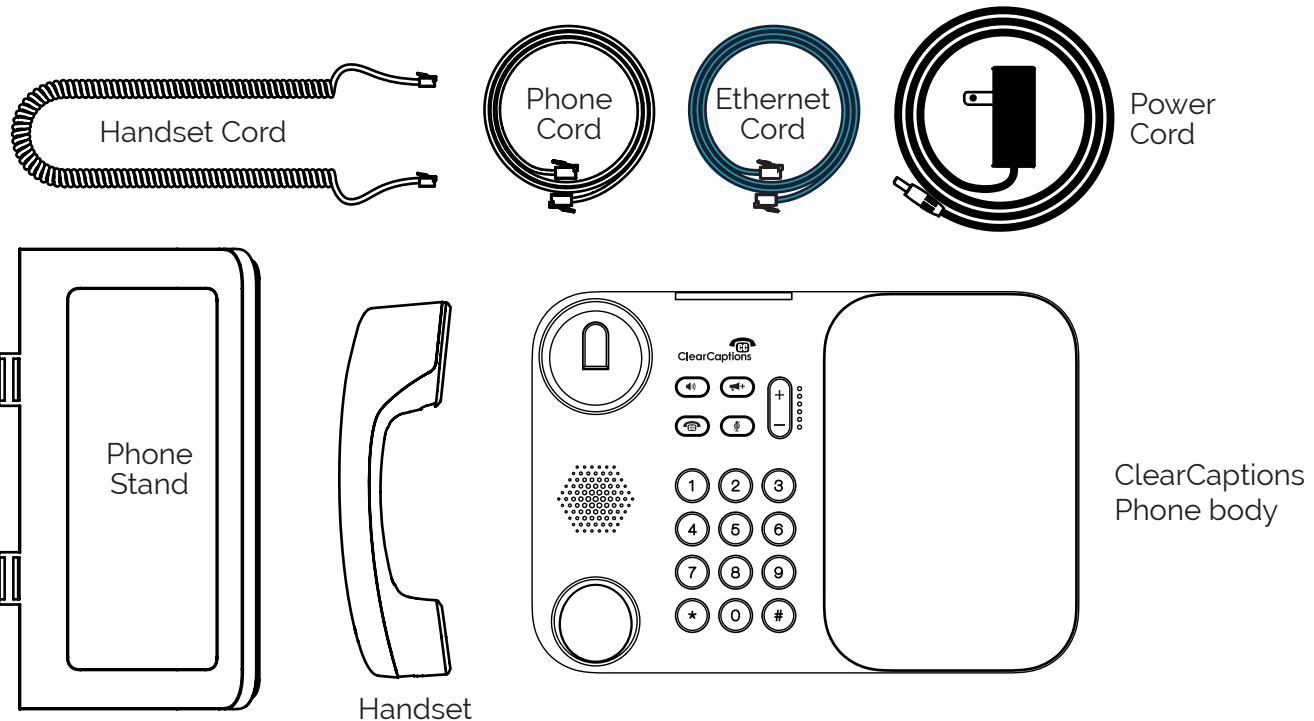


We're here to help! Call us at 866-868-8695

step


3

Insert the phone stand into one of two pairs of slots on the back of the phone body. The stand should 'snap' into place when fully inserted.

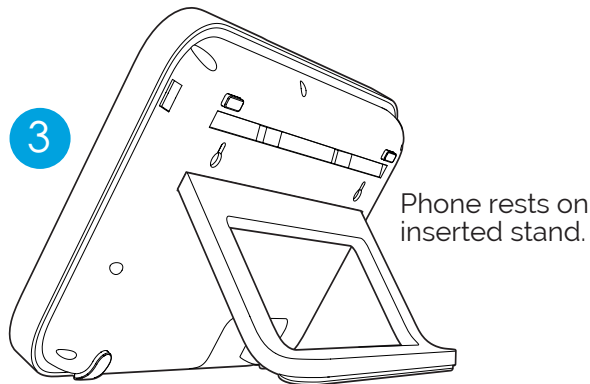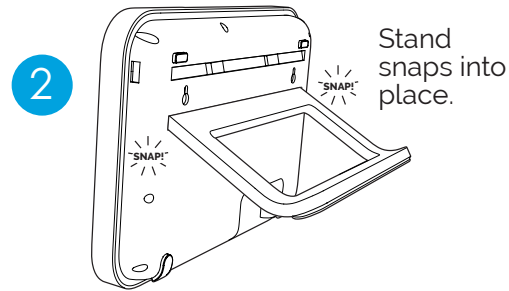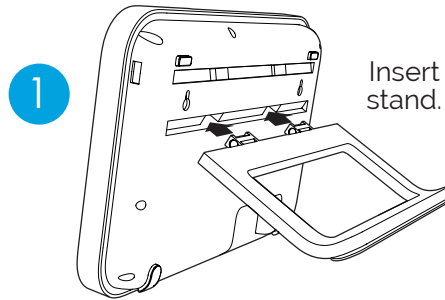

step 4

Remove the protective cover from the display screen.

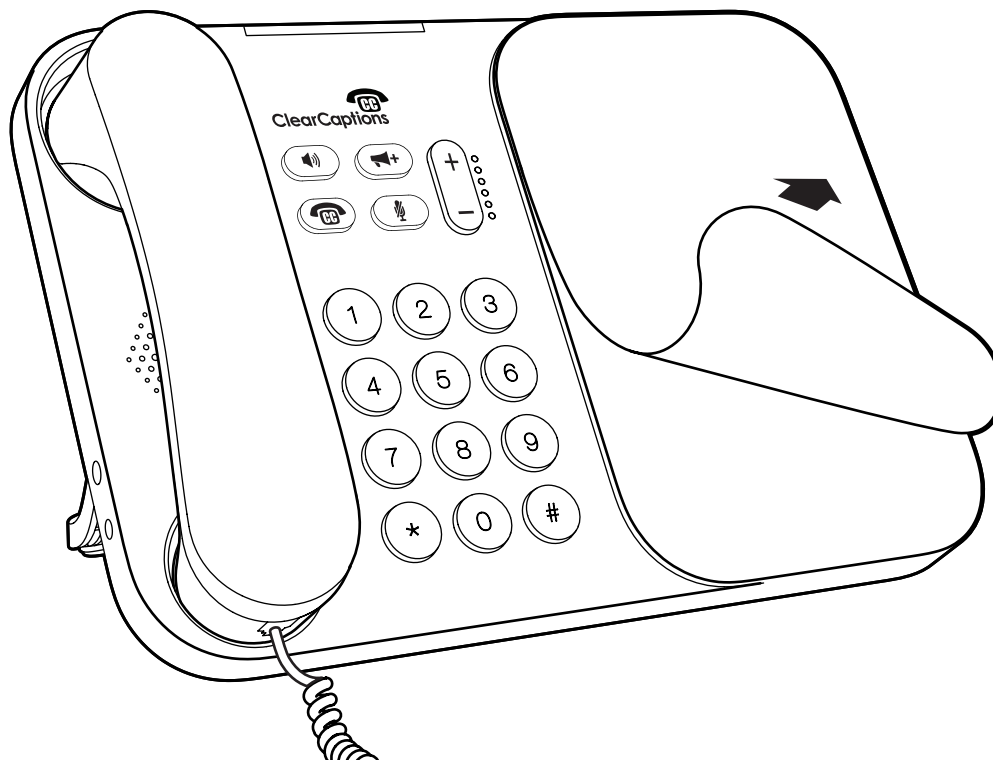


We're here to help! Call us at 866-868-8695

step

5


Plug the black power cord into the rear of the phone body. Insert the other end into an electrical outlet.

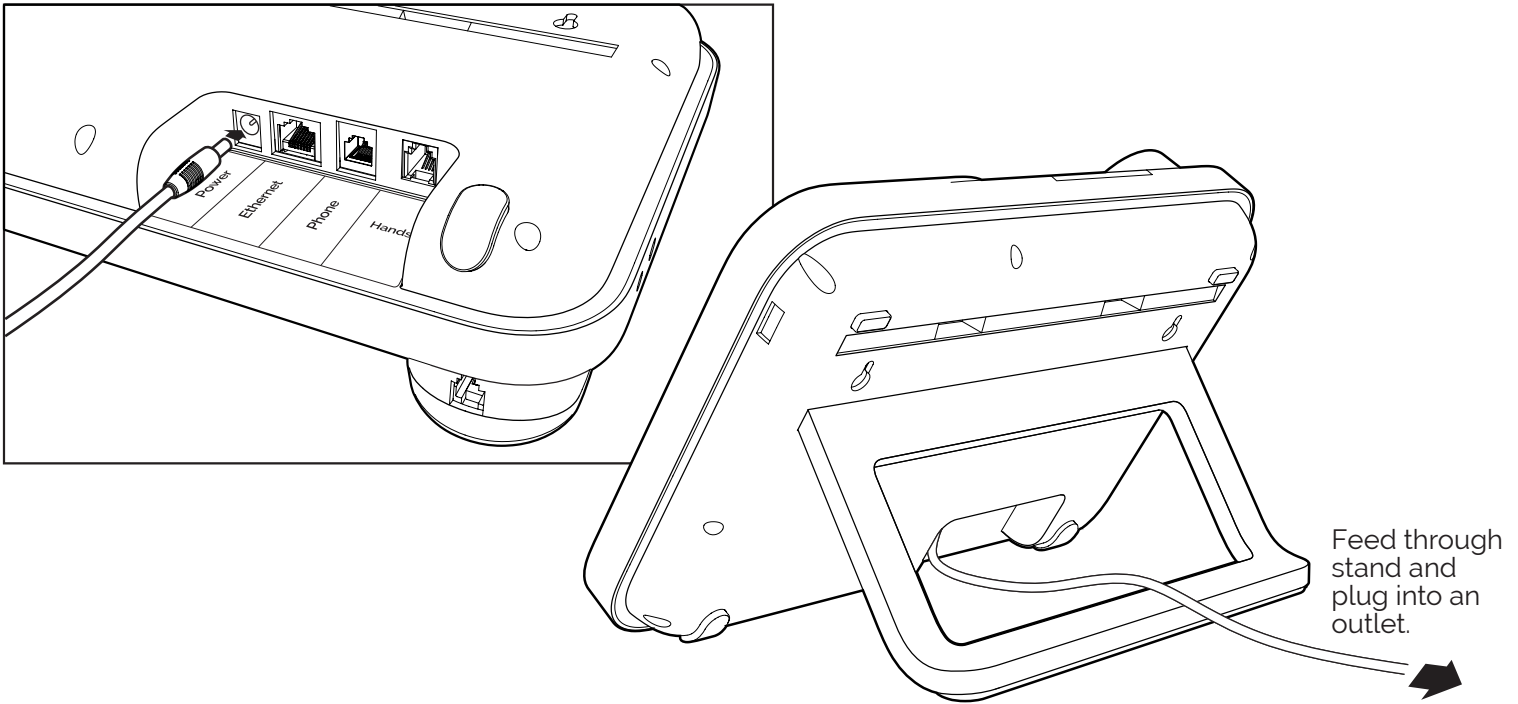


step 6

Your ClearCaptions Phone will guide you through the last few steps.

The ClearCaptions Phone has a touch screen. When you see the "Start" button, tap it.

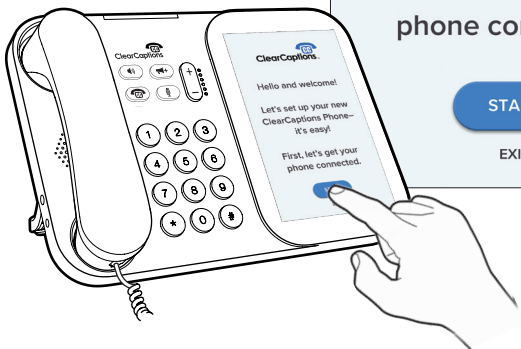



ClearCaptions™

Hello and welcome!

Let's set up your new ClearCaptions Phone—
it's easy!

First, let's get your
phone connected.

START

EXIT

PHONE LINE




Plug in phone line then tap NEXT.

NEXT

EXIT

If using landline service, plug in phone line. When you're finished with this step, tap the "Next" button.

REGISTRATION



Login to your
ClearCaptions account

Enter email or phone

Enter password

☐ SHOW PASSWORD

LOGIN

FORGOT PASSWORD | REQUEST ACCOUNT

EXIT

Log in when asked.

We're here to help! Call us at 866-868-8695


ClearCaptions™

step

7

That's it! Now, you can place your first captioned phone call.

Check marks tell you where you are in the set up process.



Checking...

- ☒ Network
- ☒ Service
- ☒ Software
- ☐ Registration
- ☒ Phone Line



Congratulations! Your new ClearCaptions Phone is ready to use.

TOUR the features or tap START to use your phone.

TOUR

START



*Need help?
Tap the
ClearCaptions
icon in Contacts or
Favorites, or dial
866-868-8695.*

When you get to the end of the set up process, you can take a tour of the phone or just get started.

